IT - Pro Pilot N.H.A. - Pro Pilot H.A.

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Pro Pilot N.H.A. - Pro Pilot H.A.

Mechanical root canal instruments

Description

Pro Pilot H.A. and Pro Pilot N.H.A. are Ni-Ti instruments available in Gold, Blue and Coated, with thermal activation. They offer excellent flexibility by easily adapting to the anatomy of most root canals. Instruments with progressively variable taper allow you to work with greater confidence in establishing the glide path.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path creation.

Measures

PRO PILOT NHA							
	∅	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	144900551S	144900561S	144900571S

PRO PILOT Heat Activation							
	∅	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	144900751S	144900761S	144900771S

PRO PILOT Heat Activation Gold							
	∅	%	N/cm	RPM	21 mm	25 mm	31 mm
○ 016	15	2-6	2,5	300	144900660S	144900670S	144900680S

Packaging

Sterile device in blister pack of 6 pieces.

Recommended procedure

Operating protocol:

- Locate the canal.
- Use a coronal portion opening tool (Pro Flayre).
- Probe with an ISO 010 or 015 manual file.
- Work gently with the Pro Pilot H.A. or Pro Pilot N.H.A. rotary instrument (ISO 016) in order to obtain a reproducible glide path before canal widening and to reduce the workload of the shaping instruments and risk of breakage.
- Flush the canal using sodium hypochlorite (Endo Clor) and viscous chelator as lubricant (Canal Prep).
- Then follow the operative sequence most suitable for the case to be treated.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

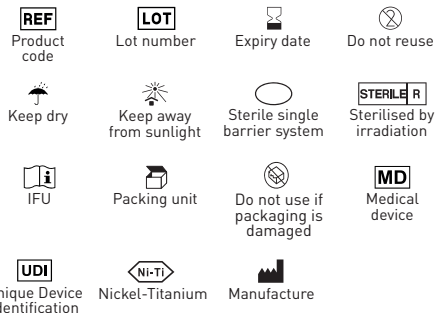
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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