IT - Pro Race

Strumenti canalari meccanici

Descrizione

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EN - Pro Race

Mechanical root canal instruments

Description

Pro Race root canal instruments are made of Ni-Ti New-Wire technology, which gives extreme flexibility and strength. Thanks to the reciprocating action, only one instrument is needed for a complete root canal treatment.

Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measures

PRO RACE					
	∅	%	21 mm	25 mm	31 mm
● R25	25	0,8	144900010S	144900020S	144900025S
● R40	40	0,6	144900011S	144900021S	144900026S
● R50	50	0,5	144900012S	144900022S	144900027S
R25-R50	--	--	144900013S	144900023S	144900028S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Prepare a root canal access.
- Select the Pro Race instrument to be used for canal preparation.
- If the canal is considered narrow use the R25 instrument. If the canal is considered medium use the R40 instrument. If the canal is considered wide use the R50 instrument.
- Preparation with the R25 instrument: estimate the length of the root canal with the help of the radiograph and adjust the stop on the instrument to approx. 2/3 of the estimated length.
- Preparation with the R40-R50 instrument: electronically determine the working length and adjust the stop on the instrument to this length.
- Irrigate the canal with E.D.T.A. (Canal Prep) and hypochlorite (Endo Clor).
- Insert the instrument into the canal and move it slowly apically, applying very light pressure; the instrument can then move forward easily. The amplitude of the up and down movements (passages) must not exceed 3 mm.
- After 3 passes clean the instrument, irrigate the canal and check its patency.
- Repeat until 2/3 of the estimated working length is reached.
- If using the R25 instrument, determine the working length electronically. If you are using the R40 and R50 instrument, after preparing 2/3 of the working length. If using the R40 and R50 instrument, after preparing 2/3 of the channel length, electronically check the working length initially established.
- Subsequently, recapitulate until the entire working length is reached.
- Determine the apical diameter by inserting a hand instrument ISO higher than the Pro Race instrument inserted up to the working length. If the hand instrument is inserted up to 0.7 mm before the working length, no further extension of the apical area is required. If the hand instrument can be inserted up to the working length, it is usually necessary to finish the preparation with a larger Pro Race instrument.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

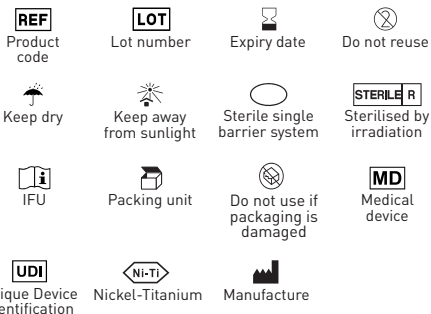
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained there in may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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