IT - Pro SH

Strumenti canalari meccanici

Descrizione

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EN - Pro SH

Mechanical root canal instruments

Description

Pro SH root canal instruments are made with New-Wire Ni-Ti technology and are characterised by high cutting capacity and flexibility.

Indications

Only to be used in dental practices or in hospitals by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measures

PRO SH					
	RPM	21 mm		25 mm	
		4%	6%	4%	6%
Ø 20	150-300	144900150S	144900160S	144900153S	144900163S
Ø 25	150-300	144900151S	144900161S	144900154S	144900164S
Ø 30	150-300	144900152S	144900162S	144900155S	144900165S
Ass.	150-300	144900156S	144900166S	144900157S	144900167S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met and determine the working length.
- Flush the canal using sodium hypochlorite (Endo Clor) and viscous chelator as lubricant (Canal Prep).
- For shaping, use shaping tool D1 until the working length determined previously is reached.
- Use shaping tool D2 and reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- If further enlargement of the apical foramen is necessary, use the appropriate finishing instrument (M2, M3, M4 and M5).

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

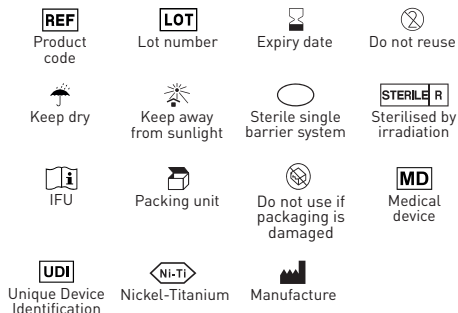
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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