IT - Pro Shaper Brown

Strumenti canalari meccanici

Descrizione

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EN - Pro Shaper Brown

Mechanical root canal instruments

Description

Pro Shaper Brown root canal instruments are made with Ni-Ti Evo Brown technology, which gives greater elasticity and superior strength compared to normal Ni-Ti instruments. This special alloy This special alloy guarantees extraordinary resistance to cyclic fatigue due to the innovative heat treatment process, as well as greater flexibility. heat treatment process, as well as greater flexibility, ensuring that the canal morphology is maintained.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

Shaping and cleaning of root canals.

Measures

PRO SHAPER Brown							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
DX	19	3,5-19	3/4	150-300	19 mm REF: 1449003405		
● D1	17	0,2-11	3/4	150-300	1449003415	1449003515	1449003615
○ D2	20	0,4-11,5	1/1,5	150-300	1449003425	1449003525	1449003625
● M1	20	0,7	1/1,5	150-300	1449003435	1449003535	1449003635
● M2	25	0,8	2,5	150-300	1449003445	1449003545	1449003645
● M3	30	0,9	2,5	150-300	1449003455	1449003555	1449003655
●● M4	40	0,6	2,5	150-300	1449003465	1449003565	1449003665
●● M5	50	0,5	2,5	150-300	1449003475	1449003575	1449003675
DX-M3	--	--	--	150-300	1449003485	1449003585	1449003685

Packaging

Sterile device in blister pack of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Locate the canal.
- Use the DX opening tool to widen the canal.
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is encountered resistance.
- Flush the canal using sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep).
- For shaping, use the D1 instrument until the working length determined previously determined.
- Use shaping tool D2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool M1 until the working length is reached.
- Measure the channel using a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument appropriate finishing instrument (M2, M3) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dental slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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