IT - Pro Shaper Remove

Strumenti canalari meccanici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Pro Shaper Remove

Mechanical root canal instruments

Description

Pro Shaper Remove root canal instruments are instruments made of Ni-Ti indicated for the removal of filling materials such as gutta-percha, cements or other filling material, prior to remodelling the root canal. There are three instruments, each for each coronal third.

Indications

For use only in a dental practice or hospital environment by qualified personnel.

Field of application

Shaping and cleaning of root canals.

Measurements

PRO SHAPER REMOVE							
	∅	%	N/cm	RPM	16 mm	18 mm	22 mm
○ D1	30	9	2,5	150-300	1449001205	-	-
○ D2	25	8	2,5	150-300	-	1449001315	-
○ D3	20	7	2,5	150-300	-	-	1449001425
D1-D3	--	--	--	--	1449001235		

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

- Isolate the operating field with rubber dam.
- Introduce a few drops of chemical softening solution for gutta-percha or other material to be removed into the pulp chamber.
- With the help of the root canal instrument D1 remove the material in the coronal third.
- Use the D2 instrument to remove material in the middle third.
- Use D3 to remove in the apical third.
- Remove the application regularly and rinse alternately with sodium hypochlorite (Endo Clor).

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials. It is recommended to always use surgical gloves for personal protection against bacterial

contamination. Failure to do so may result in cross-infection.

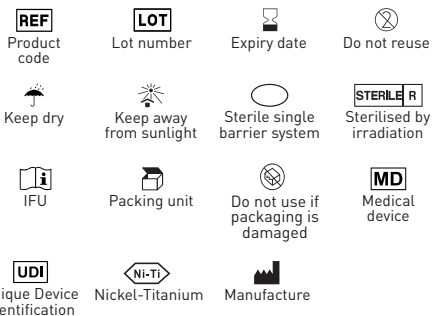
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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