IT - Pro True

Strumenti canalari meccanici

Descrizione

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EN - Pro True

Mechanical root canal instruments

Description

Pro True root canal instruments are suitable for treating and finishing most root canals and are particularly suitable in the simultaneous technique, without the need for any early enlargement of the coronal third. It consists of 4 instruments used in progression that allow the canal to be prepared up to the apex without substantially changing the morphology of the canal itself.

Indications

For use only in a dental office or hospital setting by qualified personnel.

Field of application.

Shaping and cleaning of root canals.

Measurements

PRO TRUE								
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm	
10/04	10	04	1,2	280	144900070S	144900085S	144900055S	
15/05	15	05	1,3	280	144900071S	144900086S	144900056S	
20/06	20	06	2,1	250-350	144900072S	144900087S	144900057S	
25/06	25	06	2,3	250-350	144900073S	144900088S	144900058S	
30/05	30	05	1,2	250-350	144900074S	144900089S	144900059S	
30/06	30	06	1,2	250-350	144900081S	144900096S	144900066S	
35/04	35	04	1,2	250-350	144900075S	144900090S	144900060S	
35/06	35	06	1,2	250-350	144900082S	144900097S	144900067S	
40/04	40	04	1,6	250-350	144900076S	144900091S	144900061S	
40/06	40	06	1,5	250-350	144900083S	144900098S	144900068S	
10/20	--	--	--	250-350	144900084S	144900099S	144900069S	

Packaging

Sterile device in blister packs of 6 pieces (assorted or single sizes).

Recommended procedure.

Operative protocol:

- After exploring the canal up to the apex with an ISO 010 manual file, continue with the following sequence of instruments, which are used from the smallest to the largest: 10/04, 15/05, 20/06, 25/06. This sequence allows a gradual enlargement of the entire canal in an apico-coronal manner and is valid for all canal types. It allows already with the first instrument, the creation of an immediate truncation up to the apex.
- If you feel a slowdown in the progression of the instrument, retract it 1-2 mm to make it first work passively in "brushing" action, selectively remove interference and let it advance further toward the foramen. If necessary, that is, if obstruction is again felt, the movement will be repeated.
- In the presence of canals where it was possible to easily reach the apex with an ISO 015 or 020 manual file, you may begin with the instrument having the tip diameter corresponding to that of the manual file that reached the apex.
- If it is determined that the apex measures more than 025, the basic sequence of the first four instruments can be joined by instruments of gauge 030, 035 or 040 with a taper of .06, which will ensure adequate preparations for particularly wide canals in a logic of progressive flaring proportional to the apical opening.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

Storage

The blister should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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