IT - Rotoli e Buste autosigillanti

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Descrizione

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For sterilization

Description

Self-sealing rolls and pouches are suitable for packaging medical devices that need to undergo a sterilization process.

They are made with a multilayer PET/PP copolymer film and medical grade paper. By changing color, Class I process indicators for steam and ethylene oxide provide visual confirmation that sterilization conditions have been achieved.

Indicator color change

The rolls and pouches, are equipped with Class I process indicators printed with non-toxic water-based ink. The indicators comply with ISO 11140-1, and provide clear and accurate color change after sterilization.

- **Class I steam indicator**

The color changes from **light blue** to **dark green** after being exposed to the steam sterilization process.

- **Ethyl oxide indicator class I**

The color changes from **pink** to **orange** after being exposed to the ethylene oxide sterilization process.

- **Class I formaldehyde indicator**

The color changes from **green** to **yellow** after being exposed to the formaldehyde sterilization process.

Precautions and Warnings

- The devices are for single use only. Do not reuse.
- Sterilization pouches are ready for use. Rolls, on the other hand, must be cut to the desired length.
- Do not use if the pouch is damaged.
- Assess the size of the device to be sterilized and form a pouch by sealing one of the two open ends of the roll portion with a calibrated sealing device and through a validated sealing process.
- The recommended temperature for sealing is between 170°C and 190°C, for an operation duration of 1-6 seconds.
- Cover the ends of the instruments with a protector to avoid punctures and cuts and insert the device to be sterilized into the pouch. Avoid filling the latter more than 75 percent of their capacity and leave around the instrument a minimum space of 1.5 cm from the edges to avoid excessive tension on the sealed parts.
- Before closing the envelope, remove as much air as possible from the inside of the envelope.
- Seal the open end of the envelope with a calibrated sealing device and through a validated sealing process.
- During use, avoid stress on the sealed parts and fill the envelope by holding it horizontally on a flat surface.
- When the pouches are loaded into the sterilizer, place them in the basket with the foil sides facing the foil sides and the paper sides facing the paper sides.
- The pouches should be placed in the basket upright and not too close together, that is, at a distance that allows one hand to pass through.
- The color change highlights that the package has undergone the sterilization process. Please be

advised that the change in color of the process indicator, does not guarantee that the contents are sterile; it is the responsibility of the operator to verify that they have been sterilized.

- Remove the pouches from the sterilizer carefully to avoid accidental tearing and then allow them to dry for 20 minutes.
- Do not use wet or damaged pouches; if so, change the pouch and repeat the sterilization process.
- To remove the instrument from the pouch, grasp the two flaps of the pouch and open following the direction of opening.

You must report any serious incidents that have occurred in connection with the medical device we supply to the manufacturer and to the competent authority of the member state in which you are located.

Storage

- Store at room temperature, in a cool, dry place, away from direct light and excess moisture.
- Do not use after the expiration date.
- Dispose of the product according to current regulations.

Symbols

Product code

Lot number

Expiry date

Production date

Do not reuse

Keep dry

Keep away from sunlight

Do not use if packaging is damaged

Not sterile

Steam sterilization

Ethylene oxide sterilization

Medical device

Unique Device Identification

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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