Ortho Control

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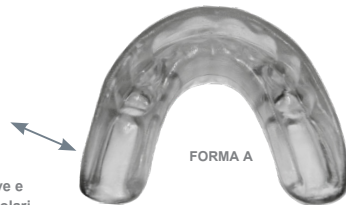
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617


e-mail: info@dental-world.it / web: www.dental-world.it

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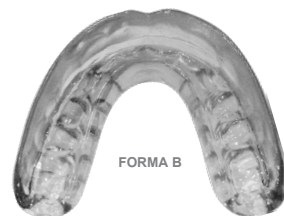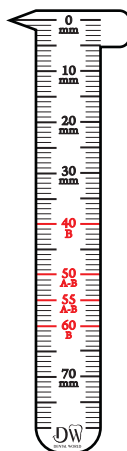
REF BB060

Ortho Control

Preformed deviced - Control - Functionality

MADE IN ITALY

Ortho Control are preformed appliances for intercepting orthodontics. They are recommended for patients from 8 to 12 years old, to guide erupting canines and bicusps and to align front teeth to solve overjet or overbite problems. Ortho Control can be also employed in adults, providing that malformation is dental and not skeletal. Moreover, they can be used as miofunctional regulators rebalancing muscular forces.

SHAPE A

Is recommended for patients from 5 to 7 years old.

SHAPE B

Is recommended for patients from 8 to years old and for the adults.

Recommended for:

- Crowding
- Overjet
- Over-bite
- I and II class malocclusion
- Accentuated spee's curve
- Hypertonic lips
- Open bite
- Rotation
- Bicuspid cross bite
- Arcate coordination

Benefits:

- Preformed for a quick placement
- Highest treatment efficacy
- Time saving
- Cost reduction
- Quick procedures
- Sterilizable
- Comfortable use

Procedures for maintenance and use:

- After use, thoroughly rinse the product under flowing water using a brush. Then, put it back in its container.
- Check the product before each use to ensure its complete integrity.
- In case of breakage or damage to the product, immediately contact your dentist.

Symbols

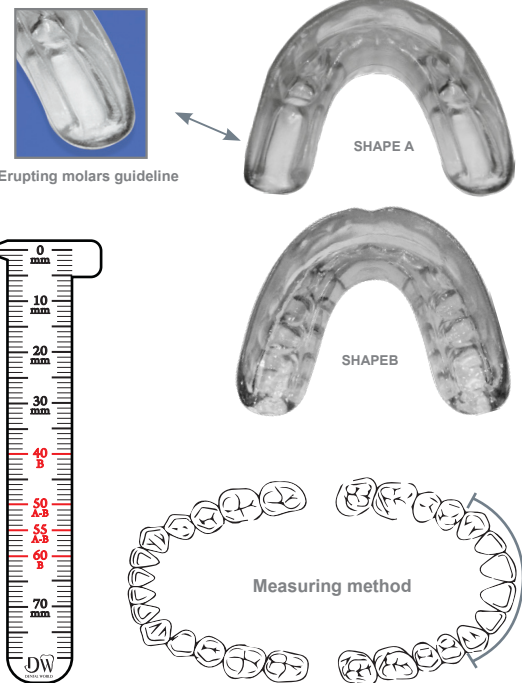
Warranty

Dental World will replace product that is proven to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

Ortho Control Kit
REF AB010

Ortho Control Refill

Shape A

REF AA050
REF AA055

Shape B

REF BB040 REF BB055
REF BB050 REF BB060