Linea Eclipse

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eclipse line

Wire for ligatures and preformed metal ligatures

Description

Steel bindings, both preformed and in coils: preformed to reduce working time, for easy anchoring and easy use; in coils for the convenience of always cutting the wire to size, thus adapting it in seconds to all cases.

Indications for use

Ligatures are used for fixation of the arch in the bracket slot, fixation of aids in and on the bracket, buccal arch tube or other accessories.

Recommended procedure

- In the case of ligature wires, cut the wire to the desired length.
- Place the ligature in use on the arch under the bracket flap and twist the end with a suitable tool until the desired fixation is achieved.
- Shorten the twisted end and bend it behind the arch near the bracket using a ligature adapter.
- Twisting the wire ligature excessively may cause deformation and breakage to the bracket, arch and ligature as well as chipping the coating.
- When twisting the ligature, be careful not to scratch the bow or covering with the tool used.

Warnings

- The product should be used by qualified professional orthodontists.
- Be careful not to injure the patient's lips and mucous membranes when cutting the ligature ends.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- In case of allergic reaction, remove the device immediately.
- The product contains nickel so in case of suspected or established allergy/sensitivity of patient do not use in any way.
- Comply with national disposal regulations.

You must report any serious incidents occurring in relation to the medical device you supply to the

manufacturer and to the competent authority of the Member State in which you are located.

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Contraindications

- Store at room temperature.
- Store properly in its packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it