Linea Maximus

Ganci e Stop

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Linea Maximus

Stop and hooks

Description

Hooks and stops are used to create an anchor or stop point on the arch.

Indications for use

Crimpable hooks and stops are passive elements used to attach and secure aids such as rubber bands, chains and springs.

Recommended procedure

- Choose the stops and hooks according to the size of the arch used. Apply the product to the arch in the desired position and secure it with crimping pliers.
- Make sure that the fastening is correct in order to avoid detachment and thus the risk of ingestion.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, car-rots, etc.
- In case of allergic reaction, remove the device immediately.
- The product contains nickel so in case of suspected or established allergy/sensitivity of patient do not use in any way.
- Comply with national disposal regulations.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

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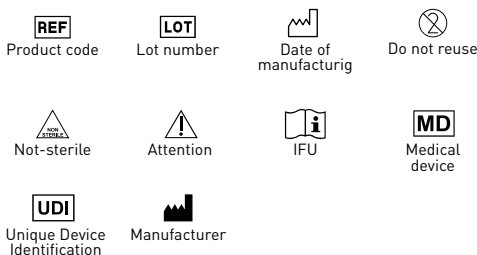
Contraindications

- Store at room temperature.
- Store properly in its packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it