Linea Eclipse

Retainer ortodontico

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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Orthodontic retainer

Description

The fixed application of orthodontic retainer on teeth over a period of time helps to maintain the result obtained with corrective braces and avoid dental recurrences. In fact, dental malocclusions resulting from crowding or recurrence may occur over the years. The retainer can be applied in different ways on teeth, going from canine to lower canine, and usually causes no discomfort or inconvenience to patients. In addition, this instrument is also used for the fixation of periodontally compromised teeth or following dental trauma to achieve dental stabilization.

Indications for use

Retention (prolonged) at the end of active treatment.

Recommended procedure

- Choose the most appropriate size by referring to the model and fit the flush segment to the lingual surfaces of the teeth.
- Use a common adhesive for bonding by observing the manufacturer's instructions.
- Bonding bases should not be touched with bare hands, as the resulting dirt can reduce their adhesion.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- In case of allergic reaction, remove the device immediately.
- The product contains nickel so in case of suspected or established allergy/sensitivity of patient do not use in any way.
- Comply with national disposal regulations.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

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Contraindications

- Store at room temperature.
- Store properly in its packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it