Linea Mimetic ed Eclipse

Bracket ortodontici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Linea Mimetic ed Eclipse

Orthodontic Bracket

Description

The bracket is a device intended for use in orthodontic treatment. The device is affixed to a tooth so that pressure, transmitted through an orthodontic wire, can be exerted on it to improve the position of the tooth.

Indications for use

Brackets can be used with traditional direct or indirect bonding methods.

For bonding procedures, follow the adhesive manufacturer's recommendations.

Recommended procedure

Ensure that the orthodontic wire is correctly and completely positioned within the slot.

To prevent fracture of the brackets, avoid direct contact of the brackets with ligature cutting instruments.

Since the bracket can cause irritation, thorough inspection by the specialist is recommended.

Removal of brackets can cause damage to enamel, so special care should be taken.

After removal of the bracket, remove the adhesive layer on the enamel with the appropriate tools.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- A light-cured adhesive is recommended for bonding the bracket to the base.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- In case of an allergic reaction, remove the orthodontic retainer immediately.
- Some bracket series contain nickel, so in case of suspected or established allergy/sensitivity of the patient, do not use in any way.

You must report any serious incidents occurring

in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.


Contraindications

Heavy smoke, tobacco and alcohol abuse, systemic blood diseases, uncontrolled diabetes.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols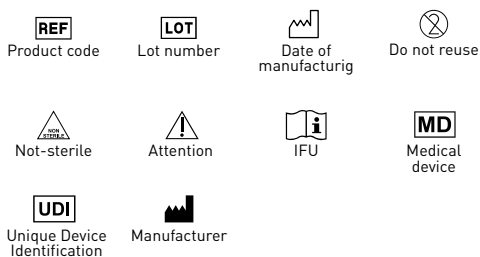

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it