Linea Eclipse

Bande ortodontiche e accessori

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Eclipse line

Orthodontic bands and accessories

Description

The molar band is a metal ring surrounding the molar; a tube is usually welded to it through which the arc necessary for treatment passes.

Lingual Sheaths are attachments made to mount palatal bars, Quad Helix or other removable appliances.

Indications for use

The objective of the band is to cement molars to exert orthodontic forces over them.

Recommended procedure

1. Bands are joined to the buccal tube or other attachments by welding to ensure that they do not come off the band during treatment.
2. Make sure that the attachments are positioned exactly and that the welds are done correctly.
3. Use an orthodontic band cement or adhesive, following the manufacturer's instructions.
4. Remove excess cement around the band and from the occlusal surface of the tooth. After cementation is complete, you can attach bows, elastics or other aids in or on the welded attachments.
5. If a band pusher is used, use it carefully to avoid slipping of the band and thus injury to the patient's mucosa.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- In case of an allergic reaction, remove the orthodontic retainer immediately.
- The product contains nickel so in case of suspected or established allergy/sensitivity of the patient do not use in any way.
- Comply with national disposal regulations.

You must report any serious incidents occurring in relation to the medical device you supply to the

manufacturer and to the competent authority of the Member State in which you are located.


Contraindications

- Inadequate oral hygiene of the patient.
- Known allergies to any of the components of the product.
- General/basic illnesses that do not permit any orthodontic treatment.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols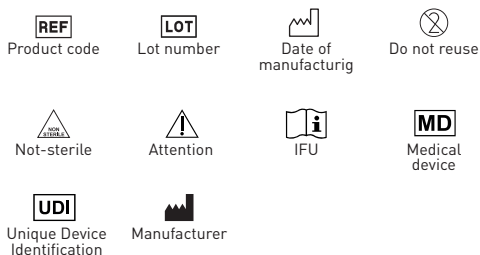

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it