# Pro Flex H.A.

## Strumenti canalari meccanici

### Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

# Pro Flex H.A.

## Mechanical canal instruments

### Description

The Pro Flex H. A. canal instruments comprise only three instruments to be used successively for shaping and finishing all kinds of canals.

### Indications

To be used only in dental clinics or in hospitals.

### Application field

Shaping and cleaning of root canals.

### Measures

PRO FLEX <i>Heat Activation</i>								
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm	
R	--	--	2,5	350	17 mm			REF: 144900399
● S1	20	0,4	2,5	250-300	144900400	144900410	144900415	
● S2	25	0,6	2,5	250-300	144900401	144900411	144900416	
● S3	35	0,4	2,5	250-300	144900402	144900412	144900417	
10-S3	--	--	2,5	--	144900403	144900413	144900418	

### Packaging

Blister 6 pcs (assorted or individual sizes).

### Recommended procedure

*Operative protocol:*

- Identify the root canal.
- Gently carry out a passive sounding of 2/3 of the coronals with an ISO 010 or 015 manual instrument (if can alternatively use mechanical files such as Pro Pat).
- Using the R instrument, create an access to the orifice of the canal and irrigate.
- For irrigation of the canal use sodium hypochlorite (Endo Clor) and viscous chelator as a lubricant (Canal Prep).
- Work carefully and gently until (obtain) a reproducible glide-path.
- Use the S2 instrument to reconfirm the working length, recapitulate (summarise) and irrigate again, especially in the curved canals.
- For large root canals use S3 instrument without any oscillatory movement.

Throughout the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E. D. T. A. (Canal Prep) in order to remove all necrotic tissue and organic component of smear layer.


### Storage

Store at room temperature, away from direct light sources.

### Preventive measures

- The instruments are recommended for single use.
- If used repeatedly they may not be efficient and/or cause inconvenience.
- Do not use the blister if opened or damaged.

### Symbols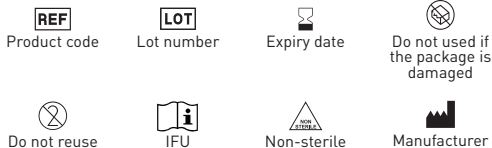

### Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: [info@dental-world.it](mailto:info@dental-world.it) / web: [www.dental-world.it](http://www.dental-world.it)