# IT - Pro Shaper 2 Evo Brown

## Strumenti canalari meccanici

### Descrizione

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# EN - Pro Shaper 2 Evo Brown

## Mechanical root canal instruments

### Description

Pro Shaper 2 Evo Brown root canal instruments are made with Ni-Ti Evo Brown technology that offers a simplified solution capable of treating a wide range of root canals. Ni-Ti Evo Brown, ensures strong and elastic instruments that harmoniously follow even the most pronounced curves without neglecting the penetrative strength of the instruments.

### Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

### Field of application

Shaping and cleaning of root canals.

### Measures

Pro Shaper 2 Evo Brown							
	Ø	%	N/cm	RPM	21 mm	25 mm	31 mm
● X1	17	0,4	2;5,2	300	144900300	144900310	144900320
● X2	25	0,5	2;5,2	300	144900301	144900311	144900321
● X3	35	0,7	2;5,2	300	144900302	144900312	144900322
● X4	40	0,6	2;5,2	300	144900303	144900313	144900323
● X5	50	0,6	2;5,2	300	144900304	144900314	144900324
Assorted	--	--	--	--	144900305	144900315	144900325

### Packaging

Non-sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

- Locate the canal.
- Use an opening tool to widen the canal or mechanically driven glide-path files (Pro Pat).
- Probe the canal with an ISO 010 or ISO 015 hand file until resistance is met.
- Flush the canal using sodium hypochlorite (Endo Clor) and viscous chelator as lubricant (Canal Prep).
- For shaping, use the X1 shaping tool until the working length determined beforehand is reached.
- Use shaping tool X2 to reconfirm the working length.
- Check the working length again by probing with an ISO 010/015 tool.
- Use the finishing tool X3 until the working length is reached.
- Measure the channel with a manual file.
- If further widening of the apical foramen is necessary, use the appropriate finishing instrument (X4, X5) according to the working length.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal slurry.

### Storage

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

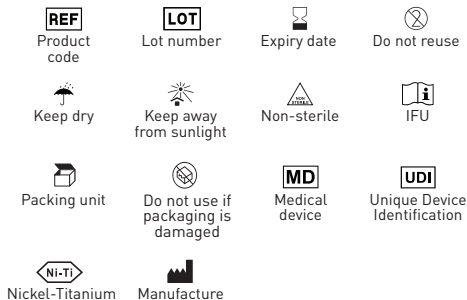
### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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