IT - Flexo File

Strumenti canalari manuali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Flexo File

Manual root canal instruments

Description

Flexo File are super-flexible steel files indicated for the preparation of irregular canals.

Indications

Only to be used in a dental practice or hospital environment by qualified personnel.

Field of application

For catheterisation and shape definition.

Measures

FLEXO FILE		
	25mm	25mm
○ 15	144517182S	144517242S
● 20	144517192S	144517252S
● 25	144517202S	144517262S
● 30	144517212S	144517272S
● 35	144517222S	144517282S
● 40	144517232S	144517292S
15-40	144550712S	144550732S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

Use the Flexo File for opening and shaping the root canal. Made of stainless steel, they provide greater flexibility for root canal cleaning. The triangular section facilitates the removal of residues. Work in quarter rotation, then withdraw the instrument.

During the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

The blister pack should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

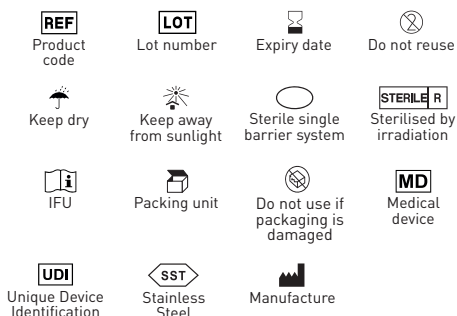
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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