IT - Spreader SS e Ni-Ti

Strumenti canalari manuali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Spreader SS e Ni-Ti

Manual root canal instruments

Description

Spreader are steel and nickel titanium instruments with a conical shape and a sharp tip. Suitable for lateral condensation of gutta-percha.

Indications

For use in dental practices or hospitals by qualified personnel only.

Field of application

For lateral condensation of gutta-percha.

Measures

SPREADER - Stainless Steel			SPREADER - Nickel Titanium	
	21mm	25mm		25mm
○ 15	144513142S	144513242S	○ 15	144443242S
● 20	144513112S	144513212S	● 20	144443212S
● 25	144513152S	144513252S	● 25	144443252S
● 30	144513122S	144513222S	● 30	144443222S
● 35	144513162S	144513262S	● 35	144443262S
● 30	144513132S	144513232S	● 30	144443232S
15-40	144550832S	144550842S	15-40	144450842S

Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

Recommended procedure

Operating protocol:

Insert a master cone of appropriate size and taper until resistance is felt to insertion. The master taper is chosen in relation to the instrument used to shape the canal. Check radiographically for correct positioning and remove the cone. Mix the cement canal cement and insert it into the canal, then introduce the master cone into the canal again. Place the master cone, which is laterally compacted using the Spreader.

During the endodontic sequence, irrigate repeatedly with hypochlorite (Endo Clor) and E.D.T.A. (Canal Prep) in order to remove all necrotic tissue and the organic component of the dentinal disease.

Storage

The blister pack should only be opened at the time of surgery and under controlled aseptic conditions. Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

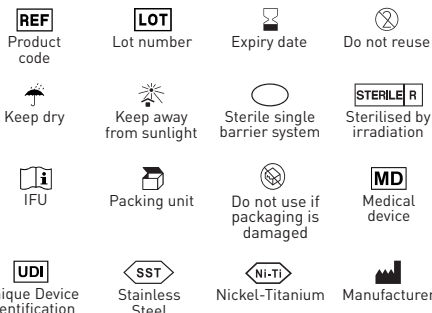
Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device you have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for careless use.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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